

# Ma Ma Swaddle

8090

“Patent Pending”



Precious Image™  
creations

## The Ma Ma Swaddle

for

Skin to Skin Care

AKA Kangaroo Care

- Safely Hold Your Full Term & Premie Baby
- Discreetly Breastfeed Hands Free
- Pump Hands free with Baby or without Baby
- Perfect for Standing Transfer
- Safely hold Premie & support Premie lines
- Can be worn as a circular wrap or over the shoulder
- Easy Fit—2 sizes

### Full Term & Premie Infants

#### Full Term Use



Breastfeed  
Hands Free

Discreetly

#### NICU Use



Pump Hands Free



Comfortably Secure Lines



Easily Secure Lines

Comfortably Hold Your Baby Skin to Skin

Two sizes:

Will adjust to fit:

Medium — S-M-L

Extra Large—XL-XXL-XXXL

#### Worn as Circular Wrap



Circular Wrap



Pump Hands Free



Pump Hands with  
your Baby



Breastfeed Hands Free

View the videos on our website  
“Kangaroo Mother Care for Your Newborn”

[www.preciousimagecreations.com](http://www.preciousimagecreations.com)



# Ma Ma Swaddle for Baby



Back View



Wrap one side over Infant and up on shoulder



Wrap second side over Infant and up on shoulder.

Center Swaddle in back and pull to front. Lay folded ties back on shoulder.



Cross ties & bring to front.



Position ties under baby's bottom and tie in a square knot to secure.



To Breastfeed, expand fabric on the side for feeding, by pulling on the tie to expand. Slide Infant to position



For Coverage when Breastfeeding, pull up one section

## Pump Hands Free



To pump, slide flanges between the two front sections and adjust.

## Worn Strapless as a Wrap Around



When worn strapless, pull one section up on shoulder to support baby when feeding.



To pump hands free slide flanges between layers.

# Ma Ma Swaddle NICU Instructions for Use with the Preemie



Back



Center Swaddle in back, pull sides to front. Fold ties back and lay on shoulders

Place baby between breast. Tubing can be placed on folded tie on shoulder.

Bring tie from unvented side over to shoulder and slide under venting and tubing.



Bring the second tie over baby and up on opposite shoulder.



Cross ties in back and bring to front.



Tie in a slip knot, so you will be able to release front with one pull on the tie.



Vent is then secured between the two layers. The tubing can lay on the tie on the shoulder.

## Use Line Holder when needed and Pump Hands Free



With Velcro strips open, slide the line holder under the tubing, lines and vent. Close strips and secure. The bottom of the line holder is placed between the two layers when worn without straps.



The tie is pulled over the bottom of the line holder, then up over the shoulder. Cross ties in back and bring ties to the front. The tie secures edges of the line holder.



To pump, slide flanges between the two front sections and adjust.