



The Second Skin Swaddle

Featuring the

The Moderate Hold

Add The Second Section of the Swaddle

- There are two identical sections to the Second Skin Swaddle. A Single Section is used for Labor and Delivery a second section is added for the moderate hold.
- The design and flexibility of the Swaddle provides all that is needed for Labor/ Delivery and Postpartum while mom is in bed.
- The Second Section gives added support and is needed for moving about with your baby.. The First Section is pulled to your shoulder like a Sling. The Second Section is pulled over Mom's torso and baby for coverage of mom and added support for baby.
- This allows you to hold your baby in the preferred upright position for Skin to Skin Contact and easily move to the breastfeeding position or simply gaze at your baby. And be hands free. To change sides for feeding move the sling to the opposite shoulder.
- The ARC Design of the Swaddle comfortably secures baby.
- If Mom needs to pump hands free that is also an option with the two sections.